



Motorized Inverted Microscope for ICSI/IMSI

ECLIPSE *Ti2*-1

Streamlining the ICSI workflow

In ICSI, speed and preciseness are indispensable, and microscopes that support ICSI must be extremely reliable and enable accurate control with simple operations. The ECLIPSE Ti2-I realizes motorized switching of observation modes to meet embryologists' needs. It simplifies the complicated microscope control of conventional models, and contributes to improvement of productivity with intuitive operability. The ECLIPSE Ti2-I provides new solutions to optimize ICSI workflows and supports an enhanced user experience.

Improving Workflow Efficiency

The ECLIPSE Ti2-I includes new features to simplify your workflow and streamline your processes

Easy Operation

Microscope control with minimal operation to save you time at the microscope

Unique Design to Increase Confidence

Intuitive design with clear visibility of the microscope settings for your convenience

High Optical Quality

Nikon's renowned optics provide high levels of clarity and overall optical performance




Improving Workflow Efficiency

New Nikon solution to simplify your workflow


Settings integrated into one button for your convenience

With conventional microscopes, it was necessary to operate and optimize multiple parts of the microscope for each observation mode, such as oocyte observation or sperm injection. However, with the ECLIPSE Ti2-I, microscope optical settings for switching observation modes are integrated into a single button. This can be completely programmed to user preferences; even complex microscope operations can be completed with one touch.

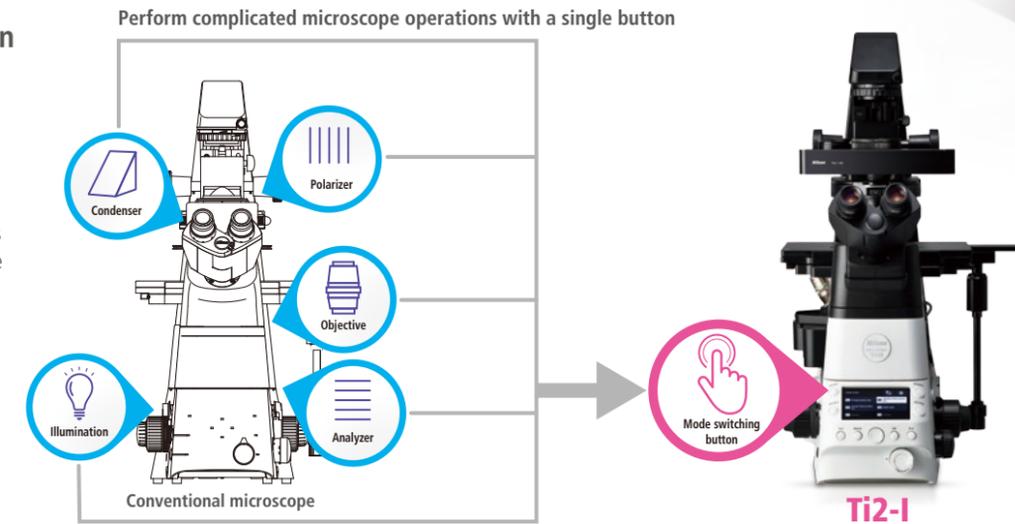

Simplified ICSI workflow

Microscopes require specific settings for each observation during assisted reproduction techniques (ART) such as ICSI and IMSI. The ECLIPSE Ti2-I requires only one button press to alter multiple settings when changing observations. This dramatically refines the workflow and improves efficiency in ICSI which requires frequent changes in observation methods.

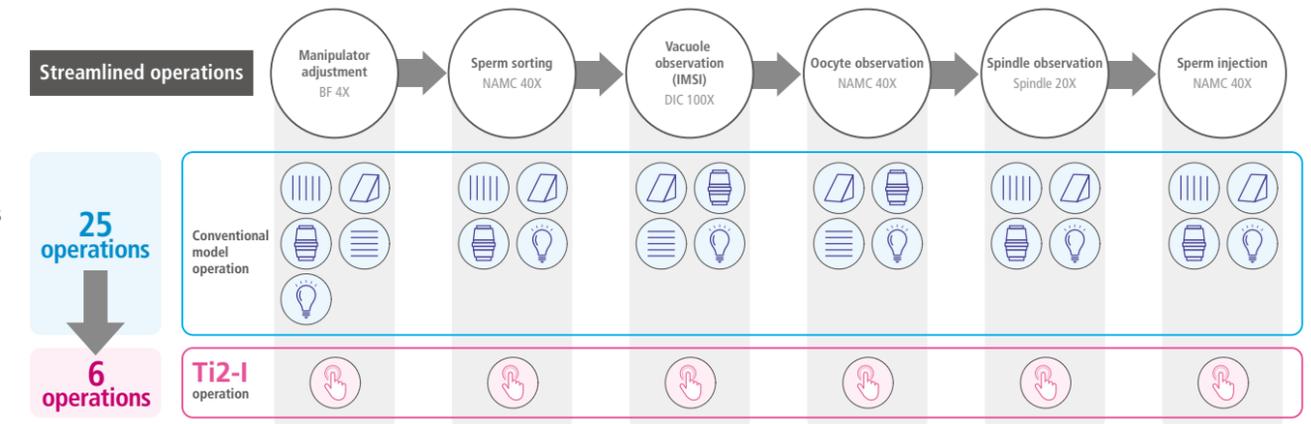

The workflow shown is an example for the purpose of illustration only.

Easy Operation

Microscope control with minimal operation



New ECLIPSE Ti2-I



Motorized observation mode switching



Automatically reproduce the adjusted brightness



Motorized observation mode switching

Control systems such as buttons are all in one place for ease of use. Motorized observation mode switching is available with just one push of a button. The microscope can be operated with minimal hand movements, allowing you to focus completely on ICSI technique.

Observation modes can be allocated to four mode buttons and two function buttons. This enables registration for observations at different magnifications, IMSI or spindle observations, and also supports the introduction of other types of observation such as laser observation.



Switch observation mode with one touch




Buttons can be operated while looking through the microscope.



Automatically reproduce the adjusted brightness


The ECLIPSE Ti2-I includes the light intensity management (LIM) function developed by Nikon. This automatically stores any changes to brightness settings which helps avoid drastic changes in brightness when switching magnification or observation mode, improving work efficiency and helping to mitigate eye strain.

Conventional model

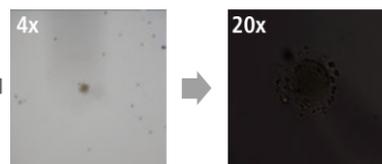

Brightness adjustment is required after changing magnification.

Ti2-I

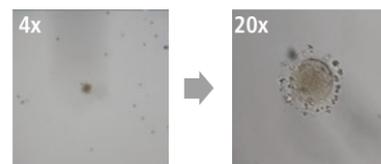

The adjusted brightness will be reproduced when changing the magnification.

Unique Design to Increase Confidence

Intuitive design with visibility of microscope settings



New ECLIPSE Ti2-I



Display showing microscope status



Alerts on setting errors



Display showing microscope status

The currently selected observation mode can be easily checked on the LCD display. The settings are clear and easy to understand for all users of the microscope. The registration settings for each mode can be confirmed using intuitive icons on the detailed information screen.




Detailed information screen



Alerts on setting errors

If the microscope settings do not match the registered configuration, warnings will be displayed. You can check on the screen where you need to make corrections. This makes it easier to notice setting errors and reduces operational mistakes.

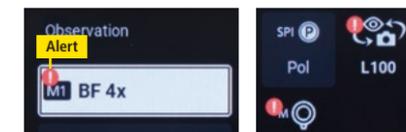

Detailed information screen



Specifications

Main Body	Optical system	CFI60 infinity optical system
	Field number	22
	Intermediate magnification	Manual switching of 1.0x/1.5x Status detection
	Output port	4 manual positions Eyepiece 100%, left 100%, right 100% Eyepiece 60%/left 40% Status detection
	Focusing unit	Coarse/fine focusing knob Coarse motion torque adjustable With refocusing mechanism Stroke: up 4 mm, down 3 mm
Tube (field number 22)		Binocular S tube, Ergonomic ER tube
Transmitted illumination	LED Lamphouse for dia illumination	High power LED
	Pillar for dia illumination	Condenser vertical stroke: 66 mm Backward tilting up to 25 degrees With field diaphragm and refocusing mechanism
Condenser		7 motorized positions (ø37 mm x 4, ø39 mm x 3)
Stage		Stroke range: adjustable in 3 levels with adjusting pin X x Y: ±57 x ±36.5 mm/±36.5 x ±25 mm/±9 x ±9 mm
Nosepiece		6 motorized positions With DIC prism slots
Cube Turret		6 motorized positions
Spindle observation system		2 motorized positions
Observation method		Brightfield, NAMC, IMSI observation*, Spindle observation*

* Compatible with glass-bottom dishes only.

This product complies with the requirements of Regulation (EU) 2017/745 and FD&C Act (US) concerning medical devices. Please contact your local distributors or in-country representative for product availability in the country concerned.

Company names and product names appearing in this brochure are their registered trademarks or trademarks.

N.B. Export of the products* in this brochure is controlled under the Japanese Foreign Exchange and Foreign Trade Law. Appropriate export procedure shall be required in case of export from Japan.

*Products: Hardware and its technical information (including software)

Monitor images are simulated.

Specifications and equipment are subject to change without any notice or obligation on the part of the manufacturer. May 2025 ©2025 NIKON CORPORATION



NIKON CORPORATION

Head office

1-5-20, Nishio, Shinagawa-ku, Tokyo 140-8601, Japan
<https://www.healthcare.nikon.com/en/>

Manufacturer

471, Nagaodai-cho, Sakae-ku, Yokohama, Kanagawa 244-8533, Japan

Nikon Instruments Inc.

1300 Walt Whitman Road, Melville, N.Y. 11747-3064, U.S.A.
phone: +1-631-547-8500; +1-800-52-NIKON (within the U.S.A. only)
fax: +1-631-547-0299
<https://www.microscope.healthcare.nikon.com/>

Nikon Europe B.V.

Stroombaan 14, 1181 VX Amstelveen, The Netherlands
phone: +31-20-7099-000
https://www.microscope.healthcare.nikon.com/en_EU/

Nikon Canada Inc.

CANADA phone: +1-905-625-9910 fax: +1-905-602-9953
Nikon France, Succursale de Nikon Europe B.V.
FRANCE phone: +33-1-4516-4516

Nikon Deutschland, Zweigniederlassung der Nikon Europe B.V.

GERMANY phone: +49-211-9414-888

Nikon Italy, Branch of Nikon Europe B.V.

ITALY phone: +39-055-300-9601

Nikon Europe B.V., Amstelveen, Zweigniederlassung Schweiz (Egg/ZH)

SWITZERLAND phone: +41-43-277-2867

Nikon UK, Branch of Nikon Europe B.V.

UNITED KINGDOM phone: +44-208-247-1717

Nikon Österreich, Zweigniederlassung der Nikon Europe B.V.

AUSTRIA phone: +43-1-972-6111


Nikon Singapore Pte. Ltd.

SINGAPORE phone: +65-6559-3651 fax: +65-6559-3668

Nikon India Private Limited

INDIA phone: +91-124-4688-500

Dimensional diagram

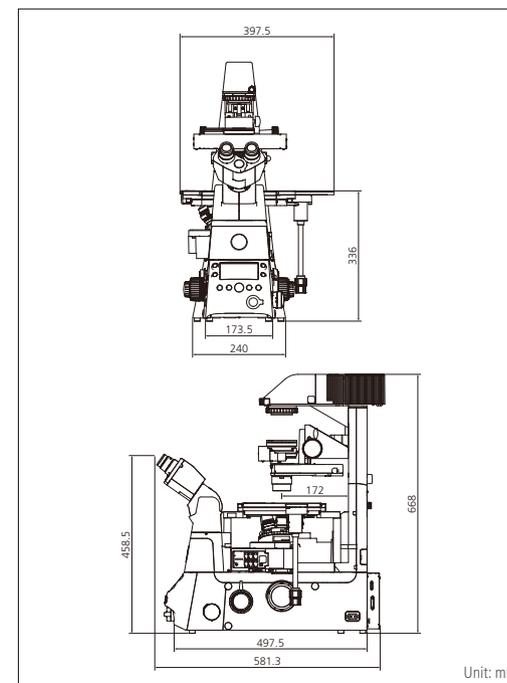

WARNING

TO ENSURE CORRECT USAGE, READ THE CORRESPONDING MANUALS CAREFULLY BEFORE USING YOUR EQUIPMENT.